

AUTOMATIC PROCESSING with Replenishment & running Water:

Patterson Dental X-Ray Solution Developer & Fixer have an excellent shelf life one year from packaging date when stored unopened & sealed at normal temperatures.

Normal processing temperature would be 28.3°C (83°F) with a cycle of 5 Minutes.

Replenishment in automatic processors varies per machine. Most processors replenish on set intervals of running time. "Rule of thumb" would be about 300 – 400 ml of replenishment per day for 30 to 50 intraoral films. Running panoramic films would require 24 ml per film more replenishment.

Developer & Fixer solutions should be changed every 3 to 4 weeks in these processors and racks should be cleaned.