MADE IN U.S.A.  
Fabriqué aux É.-U.

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N° de produit

**090-9796**

**Directions for Use:** 2 tablets make 1 gallon. Add 2 tablets to 1 gallon of warm tap water. For 2 quarts or less, use 1 tablet. Do not mix with any other cleaning or disinfecting product. Wear eye protection and rubber gloves while handling.

**For Ultrasonic Cleaners:** Depending on size of ultrasonic unit, drop either 1 or 2 tablets directly into tank. Fill with warm tap water. Water temperature should not exceed 140°F (60°C). If pre-mixing for later use, place 2 tablets in a gallon container and fill with warm tap water. Store mixed solution in a covered container until ready to use. Ultrasonically clean items for a minimum of 10 minutes. Rinse items thoroughly. Dry instruments before proceeding with the sterilization process. Replace solution when it becomes visibly cloudy, or minimally, once a day as recommended by the "ADA Recommendations for Dental Office Procedures."

**For Evacuation Systems:** At the end of each day, add 2 tablets to 1 gallon of warm tap water. Draw at least one quart of solution through each suction outlet and run system for a few minutes as recommended by the manufacturer. If heavy surgical procedures are performed, flush lines several times daily.

**For Instrument Pre-Soak:** Soak instruments in solution for at least 10 minutes, longer for tougher stains. Do not exceed temperature of 140°F.

**KEEP OUT OF REACH OF CHILDREN  
NOT FOR INGESTION**

53679042 R9/14

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Patterson Dental Supply, Inc.
MADE IN U.S.A. – Fabriqué aux É.-U.