**REORDER NUMBERS:**

Size 0: 048-1572

Size 1: 048-1580

Size 2: 048-1564

## **Patterson<sup>®</sup> PSP Barrier Envelopes**

### Instructions for Use

1. Remove a cardboard insert from left section of box.
2. Insert the phosphor plate with magnet on plate, aligned to cut-out on the cardboard insert.
3. Fold cardboard insert.
4. Remove a phosphor plate envelope from right side of box.
5. Place folded cardboard insert, with the phosphor plate, into the sleeve. Use the semi-transparent side of the envelope to make sure plate is loaded correctly.
6. Peel away White Tab and pinch opening to seal envelope.
7. Place envelope in patient's mouth and expose.
8. Bring envelope to processing area while keeping the semi-transparent side of the envelope face-down during transport to avoid lightening image prior to processing.
9. At your processing machine, flex the butterfly tabs at opening of envelope and peel down the center seam. You can now feed your plate into the processing machine.
10. Discard used envelope and cardboard insert.
11. When the plate is dispensed from your processing machine, take a fresh cardboard insert and envelope and repeat steps 1 – 9.

**Please note:** There are different size envelopes and inserts for different size sensors.

**Manufactured for:** Patterson Dental Supply, Inc.  
1031 Mendota Heights Road  
Saint Paul, MN 55120  
*Fabriqu e pour :*

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