

# Patterson<sup>®</sup> MicroBVac

## All-natural Microbial Evacuation System Cleaner Meets NEW EPA pH guidelines

### Instructions For Use:

- Super Concentrated Formula eliminates Biofilm plaque, organic solids, scum, odors, “Daily Trap Hand Cleaning” & improves vacuum suction. 100% all natural & environmentally friendly. Safe for animals, humans, and all plastic parts and pipes.
- Shake before each use.

### Twice Weekly Maintenance Protocol

At closing (per operatory) add 15ml of eVac cleaner to 500ml of warm water and mix well. Aspirate 250ml of solution into each valve (HVE & SE) one valve at a time or 500ml per operatory 2x per week.

OR

### Daily Maintenance Protocol

At closing (per operatory) add 5ml of eVac cleaner to 500ml of warm water and mix well. Aspirate 250ml of solution into each valve (HVE & SE) one valve at a time or 500ml per operatory daily.

**IMPORTANT:** Check traps monthly as daily trap cleaning will no longer be needed!

Evacuation System Cleaner can also be used in cuspidors and sinks (use as needed). Reseal lid to ensure freshness. Rinse out your dispenser/bucket after use and allow to dry.

Manufactured for:  
Fabriqué pour :  
Patterson Dental Supply, Inc.  
1031 Mendota Heights Road  
Saint Paul, MN 55120  
800.873.7683

### Eliminates Daily Trap Cleaning



Trap Before

Trap After

### Reduces Amalgam Filter Replacements



Filter Before

Filter After

### Microbes Clean Between Uses



Pipe Before

Pipe After