

Patterson Ultrasonic Cleaner PA4 & PA13Q

Application	Water To Concentrate	
• General Cleaning	40 to 1	5 Minutes
• Cement removal	10 to 1	5 – 10 Minutes
• Laboratory Polishing	10 to 1	5 Minutes
• Investments	10 to 1	5 – 10 Minutes

For best instrument care, proceed with sterilization within 30 minutes of removal from ultrasonic solution. Pre-scrubbing instruments is not required when using a high-intensity ultrasonic cleaner. Change solution daily. When using cassettes, double cleaning time.

Warning: Keep out of reach of children. Avoid contact with eyes and skin.

May cause eye and skin irritation; harmful if swallowed.

First Aid:

Eye Contact: Flush with flowing water for 30 minutes and see medical attention.

Skin contact: Wash skin with water and mild soap. Remove contaminated clothing.

If irritation persist, seek medical attention.

Ingestion: Do not induce vomiting. Give milk to drink and see medical attention.

Inhalation: Remove to fresh air. If irritation persists, seek medical attention.