



# PATTERSON<sup>®</sup> DENTAL

## Patterson<sup>®</sup> 5% Sodium Fluoride Varnish

Each mL of varnish contains 50 mg of Sodium Fluoride (equivalent to 22.6 mg fluoride ions).

### Indications For Use

Patterson<sup>®</sup> 5% Sodium Fluoride Varnish is a fluoride containing preparation for use as a cavity varnish and for the treatment of dentin hypersensitivity.

### Contraindications:

1. Known allergic reactions to fluoride.
2. Ulcerative gingivitis and stomatitis.
3. Avoid ingestion during application.
4. Known allergies to colophony or similar rosin materials.

### Precautionary Information for Dental Personnel

Use only as directed. To limit ingestion, determine the proper dosage amount for the patient. Discontinue patient's use of other prescriptive fluoride preparations (i.e. gels, rinses) for 24 hours. Patients taking fluoride supplements should discontinue use of supplements for 2-3 days following treatment.

### Warning

Contains ingredients that may be irritating to the skin and eyes. Edematous swelling has been reported only in rare instances, especially following application to extensive surfaces with fluoride varnish. Dyspnea, although extremely rare, has occurred in asthmatic people.

### Instructions for Use

1. Clean teeth and remove obvious plaque and calculus from area to be treated. For best results, remove any excess moisture from the tooth surface.
2. Open the unit-dose of Patterson<sup>®</sup> Varnish by gripping the tray and slowly peeling back the foil cover, exposing the brush applicator and varnish well.
3. Remove applicator brush and thoroughly stir the Patterson<sup>®</sup> Varnish in the well until the varnish appears to be uniformly mixed.
4. Paint a thin layer of varnish onto treatment area using broad and even strokes.
5. Let the varnish dry for approximately 10 seconds and then instruct the patient to close their mouth.
6. When dry, any excess varnish may appear "spotty" and may slide off the tooth surface. This will not affect the efficacy of the product as there is still a full layer of varnish on the tooth.
7. Dispose package and applicator after use.
8. Patient may leave immediately after treatment. Patient should be advised to not brush teeth for a minimum of 2 to 4 hours after treatment.

### NON MEDICINAL INGREDIENTS

Ethanol, Rosin, xylitol, thickener and flavors.

For professional use only



**Caution:** Consult accompanying documents.



Single Use Only

Item #	Packaging	Flavors
133-3475	50/box	Mint
133-3483	50/box	Caramel
133-3491	50/box	Cherry
133-3657	50/box	Bubblegum
133-3665	50/box	Strawberry

Manufactured for:

Patterson Dental Supply, Inc.  
1031 Mendota Heights Road  
Saint Paul, MN 55120

Made in USA

Rev.00