



Long-Lasting Strength.

The Calibra® Cement system, including Prime&Bond® NT™ Dual Cure Adhesive, provides outstanding mechanical properties to ensure the restoration you place will last. Calibra® Cement is formulated with the same resin system used in TPH®3 Restorative, one of the most popular composites in the industry.

Flexural Strength

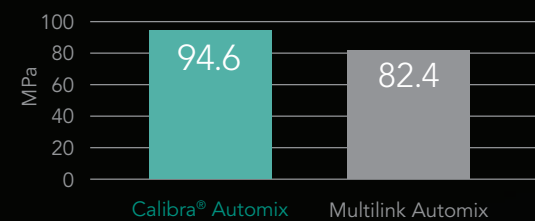


Flexural Modulus

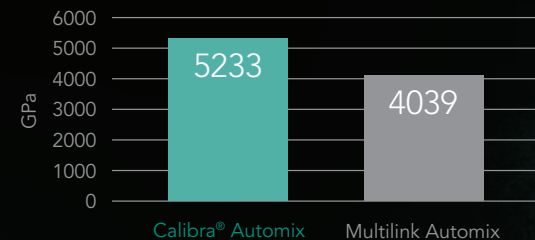


Retention Comparable to Multilink.

Keep the indirect restoration in place by combining Calibra® Cement with Prime&Bond® NT™ or Xeno® IV adhesives.

Bond Strength to Dentin

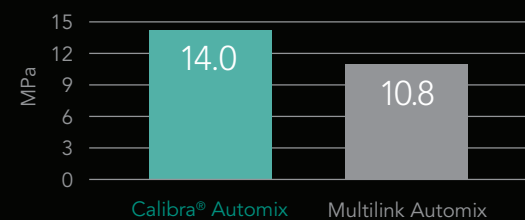

Bond Strength to e•max

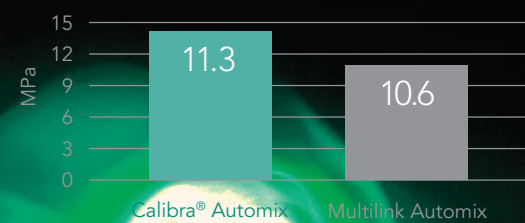

Ordering Information

607081	Dual Cure Automix Syringe Refill Package 1 Syringe (4.5g), 10 Mixing Tips	607059	Operatory Kit 2 Syringes Base – 1 each shade: Translucent, Light (2g each) 2 Syringes Catalyst – 1 each: Regular Viscosity, High Viscosity (2g each) 1 Bottle Prime & Bond® NT™ Adhesive (4.5mL) 1 Bottle Self Cure Activator (4.5mL) 1 Syringe Silane Coupling Agent (3mL) 25 Disposable Applicator Tips (Needles) 1 Mixing Stick 1 Brush Handle 40 Disposable Brush Tips 1 Dispensing Well 1 Mixing Pad 1 Light Protection Cover
607082	Light		
607083	Medium		
607084	Dark		
607085	Translucent		
	Opaque		
607086	Mixing Tip Refill Package (Clear) 50 Mixing Tips	607058	Try-In Accessory Pack 5 Syringes Try-In Paste – 1 each shade: Translucent, Light, Medium, Dark, Opaque (1.8g each)
607090	Veneer Kit 5 Syringes Base – 1 each shade: Translucent, Light, Medium, Dark, Opaque (2g each) 5 Syringes Try-in Paste – 1 each shade: Translucent, Light, Medium, Dark, Opaque (1.8g each) 1 Bottle of Prime & Bond® NT™ Adhesive (4.5mL) 1 Syringe Caulk® 34% Tooth Conditioner Gel (3mL) 1 Syringe Silane Coupling Agent (3mL) 25 Disposable Applicator Tips (Needles) 50 Flocked Applicator Tips 1 Dispensing Well 1 Light Protection Cover		Try-In Paste Refill Package 2 Syringes (1.8g each) 607071 Light 607072 Medium 607073 Dark 607074 Translucent 607075 Opaque
607060	2-Part Complete System 5 Syringes Base – 1 each shade: Translucent, Light, Medium, Dark, Opaque (2g each) 2 Syringes Catalyst – 1 each: Regular Viscosity, High Viscosity (2g each) 5 Syringes Try-In Paste – 1 each shade: Translucent, Light, Medium, Dark, Opaque (1.8g each) 1 Bottle Prime & Bond® NT™ Adhesive (4.5mL) 1 Bottle Self Cure Activator (4.5mL) 1 Syringe Silane Coupling Agent (3mL) 1 Syringe Caulk® 34% Tooth Conditioner Gel (3mL) 25 Disposable Applicator Tips (Needles) 1 Mixing Stick 1 Brush Handle 40 Disposable Brush Tips 1 Dispensing Well 1 Mixing Pad 1 Light Protection Cover		Base Refill Package 1 Syringe (2g) 607061 Light 607062 Medium 607063 Dark 607064 Translucent 607065 Opaque
			Catalyst Refill Package 1 Syringe (2g) 607068 Regular Viscosity 607069 High Viscosity
		607080	Silane Coupling Agent Refill 1 Syringe (3mL) 25 Disposable Applicator Tips (Needles) – Blue

Calibra®
Esthetic Resin Cement

www.CalibraCement.com

Finish Strong.

Never compromise on a job well done.

Call your DENTSPLY Caulk representative or visit www.CalibraCement.com

DENTSPLY
CAULK

38 West Clarke Avenue • Milford DE 19963 • 1.800.532.2855 • www.dentsply.com • www.caulk.com

All data on file. Cercon, Captek, e•max and Multilink Automix are not registered trademarks of DENTSPLY International. ©2012 DENTSPLY International. All rights reserved. Form #507910 (5/4/12)



DENTSPLY
CAULK

Calibra®
Esthetic Resin Cement

Finish Strong.

It's beautiful. It's strong. And now, with our new light cure Veneer Kit and dual cure Automix Syringe, Calibra® Cement is easier to use than ever before. So you can work faster and deliver uncompromising esthetics and strength for all your indirect restorations.



Finish Beautifully.

Calibra® Esthetic Resin Cement offers five shades to create a beautiful match for any patient. And it's Shade Stable™, so your restoration will continue to look as vibrant as the day you cemented it.

- Full spectrum of esthetic shading possibilities
- DENTSPLY Caulk's Shade Stable™ chemistry gives you the confidence that the shade you place will not shift over time



Displayed colors may not perfectly represent the actual product

Finish the Easy Way.

Our new dual barrel syringe gets the job done fast and easy. Dispense Calibra® Dual Cure Automix into indirect restorations and enjoy all the advantages of a no-compromise esthetic resin cement.

- Light cure excess cement for 10 seconds to initiate a gel phase that makes cleanup fast and simple
- Dispense, cure, cleanup and you're done

Hand-Mixing Also Provides Superb Handling.

If you prefer hand-mixing, Calibra® Cement is still offered in a choice of catalyst paste viscosities that provide the right handling characteristics for any application.

- High viscosity formula is never too fluid, so it won't run off a restoration during placement
- Regular viscosity is just the right consistency to flow underneath a crown or bridge for a perfect seat; other cements can be too fluid or not fluid enough

Finish Your Way.

Calibra® Cement has always meant simple, strong and beautiful restorations. Now, it's even easier and more versatile.

Dual Cure Automix Syringe

Complete your restoration faster, with less mess and waste. And choose the curing method that best suits the restoration: light cure or self cure.




Calibra® Dual Cure Automix Syringe

Finish Everything.

Use Calibra® Cement for virtually any cosmetic indirect restoration.

- Ceramic, porcelain, composite inlays/onlays, Maryland bridges, veneers and crowns
- All metal crowns, bridges, inlays/onlays including precious, semi-precious and non-precious metals
- PFM (porcelain fused to metal) crowns and bridges
- Cementation of prefabricated and cast posts

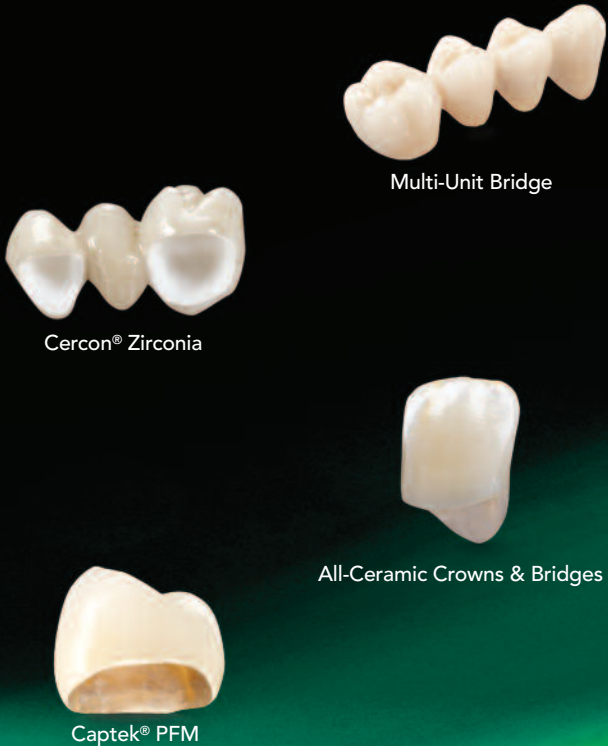

Complete Veneer Kit

Make veneers simple, beautiful and reliable with this easy-to-use kit.

- Calibra® Cement offers five shades to create a beautiful match for any patient.
- Shade Stable™ so your restoration will continue to look as vibrant as the day you cemented it.

607090 Veneer Kit Contents:

- 5 Syringes Base – 1 each shade: Translucent, Light, Medium, Dark, Opaque (2g each)
- 5 Syringes Try-in Paste – 1 each shade: Translucent, Light, Medium, Dark, Opaque (1.8g each)
- 1 Bottle of Prime & Bond® NT™ Adhesive (4.5mL)
- 1 Syringe Caulk® 34% Tooth Conditioner Gel (3mL)
- 1 Syringe Silane Coupling Agent (3mL)
- 25 Disposable Applicator Tips (Needles)
- 50 Flocked Applicator Tips
- 1 Dispensing Well
- 1 Light Protection Cover

