

Adhese® Universal

Light-curing dental adhesive

All in one click...
Up to 190 applications.



ivoclar
vivadent®
passion vision innovation

Adhese® Universal

Universal adhesion with advanced delivery

Adhese Universal is a single-component, light-curing universal adhesive for direct and indirect bonding procedures that features compatibility with all etching techniques: self-etch, selective-enamel-etch and total-etch. Adhese Universal is the only universal bonding agent available in the conventional bottle as well as the unique VivaPen® delivery for fast and convenient direct intra-oral application with minimal waste to optimize cost-effectiveness.

The VivaPen ensures consistently high bond strength because its internal locking mechanism keeps the material fresh all the way to the last drop.

- **UNIVERSAL APPLICATION** – for all bonding and etching techniques
- **EFFICIENT DELIVERY** – up to 190 single-tooth applications per VivaPen
- **PREDICTABLE RESULTS** – high bond strength and virtually no post-operative sensitivity

Universal Application

Compatible with direct and indirect bonding procedures

Adhese Universal balances hydrophilic and hydrophobic properties into one formulation making it universally compatible for direct and indirect bonding procedures. Its low film thickness (~10 microns) will help ensure accurate seating of indirect restorations.



Direct Restorations



In-Direct Restorations

Compatible with all etching techniques

Adhese Universal is well-suited for any etching protocol to ensure optimum results in all situations.



Self-Etch

Minimum treatment time with low risk of post-operative sensitivity.



Selective-Enamel-Etch

Enhances adhesion to enamel.



Total-Etch (Etch & Rinse)

Maximizes adhesion to both enamel and dentin.



Efficient Delivery

The ergonomic, pen-like design of the VivaPen combined with its angled brush cannula enhances comfort, control and speed during direct intra-oral application while reducing material waste.



Click 2 to 3 times for the adhesive to saturate the applicator brush



Apply one coat of Adhese Universal to moist prep and agitate for 20s



Disperse with air until glossy, immobile film layer results



Light cure for 10s

Virtually No Post-Operative Sensitivity



The water solvent and integrated micro-fillers used in Adhese Universal are designed to enhance penetration into the dentin tubules to stabilize and support the collagen network as well as create the formation of a homogeneous layer with only one layer of adhesive. This results in optimum sealing of the dentinal tubules to prevent movement of dentinal fluid, microleakage and post-operative sensitivity.

Ordering Information

VivaPen Delivery

663723WW	Adhese Universal Promo VivaPen (1x2ml) 100 snap-on cannulas 20 protective sleeves
665156WW	Adhese Universal VivaPen Refill (1x2ml)
627178	VivaPen Snap-On Cannulas (100 pcs.)
627492	VivaPen Protective Sleeves (300 pcs.)

Bottle Delivery

663720WW	Adhese Universal Bottle Refill (1x5g)
663721WW	Adhese Universal Bottle Refill (2x5g)
664698	Vivadent Applicators, green (50 pcs.)



100% CUSTOMER SATISFACTION
GUARANTEED!



ivoclarvivadent.com

©2014 Ivoclar Vivadent, Inc.
Ivoclar Vivadent, Adhese, VivaPen, Tetric EvoCeram and
Variolink are registered trademarks of Ivoclar Vivadent, Inc.
666153 Rev. 01 3/2014

United States

175 Pineview Drive
Amherst, New York 14228
716-691-0010
800-533-6825
Fax: 716-691-2285
www.ivoclarvivadent.com

Canada

1 - 6600 Dixie Road
Mississauga, Ontario L5T 2Y2
905-670-8499
800-263-8182
Fax: 905-670-3102
www.ivoclarvivadent.com

ivoclar
vivadent
passion vision innovation